



Instructions to install Voyager PACS Web Viewer using an Installer


1. Open a web browser and go to <https://images.infocusradiology.com.au> and click on '**Click Here**' next to '**For Windows**' (See image below).

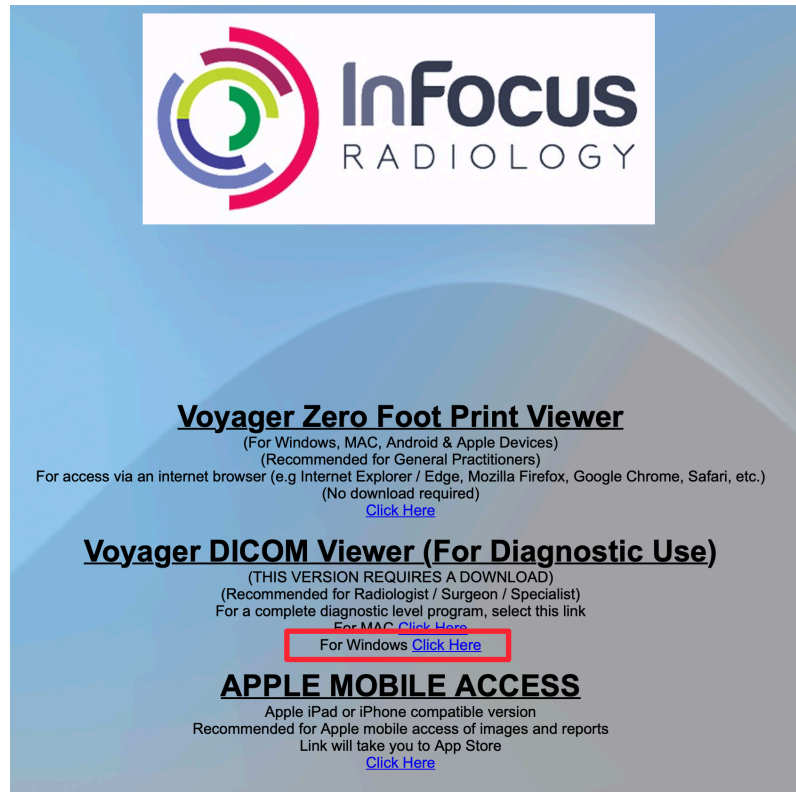


2. On the new web page, click on '**Download Now**' to download the VoyagerWebViewerSetup.exe.

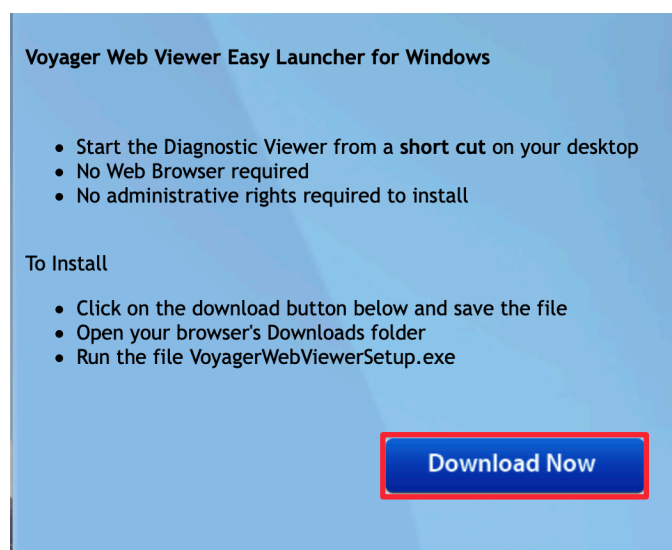


3. Once the installer is downloaded, run it. (If you get a prompt **'the publisher of VoyagerWebViewerSetup.exe couldn't be verified'**, then click **'Run'**)
4. Voyager Web Viewer Setup Wizard will open up and click **'Next'**.

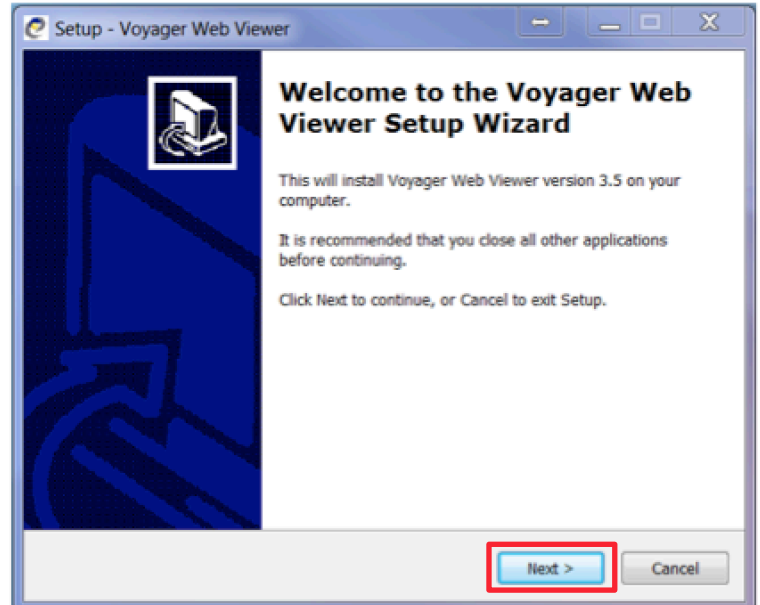

5. Under **'Select Destination Location'**, a default path will be displayed. Click **'Next'** (If you would like to install at a different location, then please **'Browse'** and select).

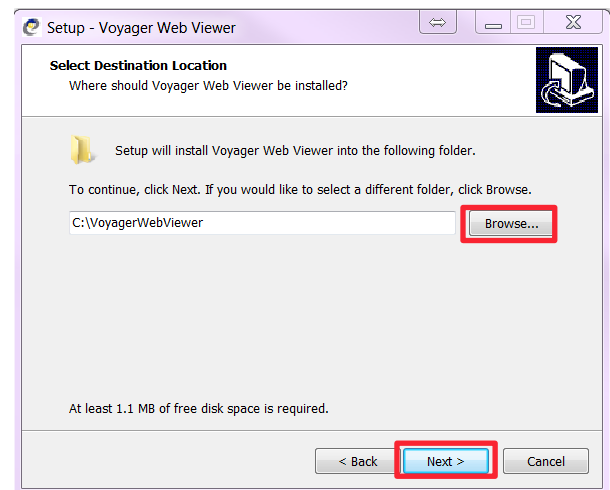

6. Under **'Select Destination Location'**, a default Start Menu Folder will be displayed. Click **'Next'** (If you would like to select a different folder, then please **'Browse'** and select).

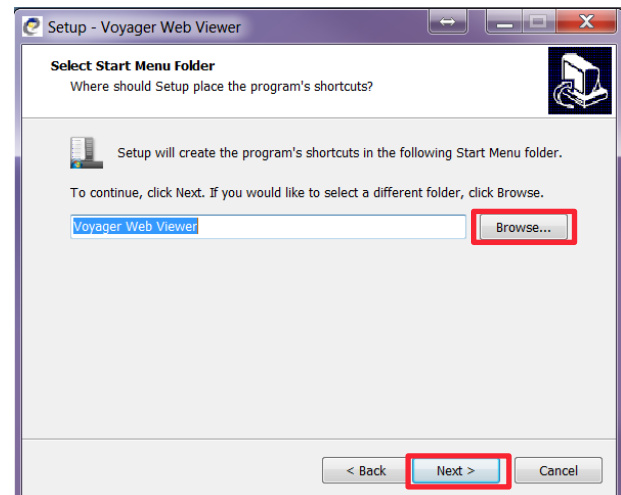

7. Under **'Select Additional Tasks'**, check the box **'Create a desktop icon'** and click **'Next'**.
8. On the next view click on **'Install'**.
9. Once the installation is complete, click **'Finish'**.
10. Go to Desktop and double click on **Voyager Web Viewer icon**.
11. Voyager Web Launcher application will open up. Put the Server Name as – <https://images.infocusradiology.com.au> and hit **'Launch Web3'**.

Files will be downloaded from Iris Imaging's Server automatically and will launch Voyager PACS Web Viewer.

12. Enter username and password to log onto Voyager PACS Web Viewer.

- END -