


Instructions to install Voyager PACS Web Viewer using an Installer

1. Open a web browser and go to <https://images.infocusradiology.com.au> and click on 'Click Here' next to 'For Windows' (See image below).

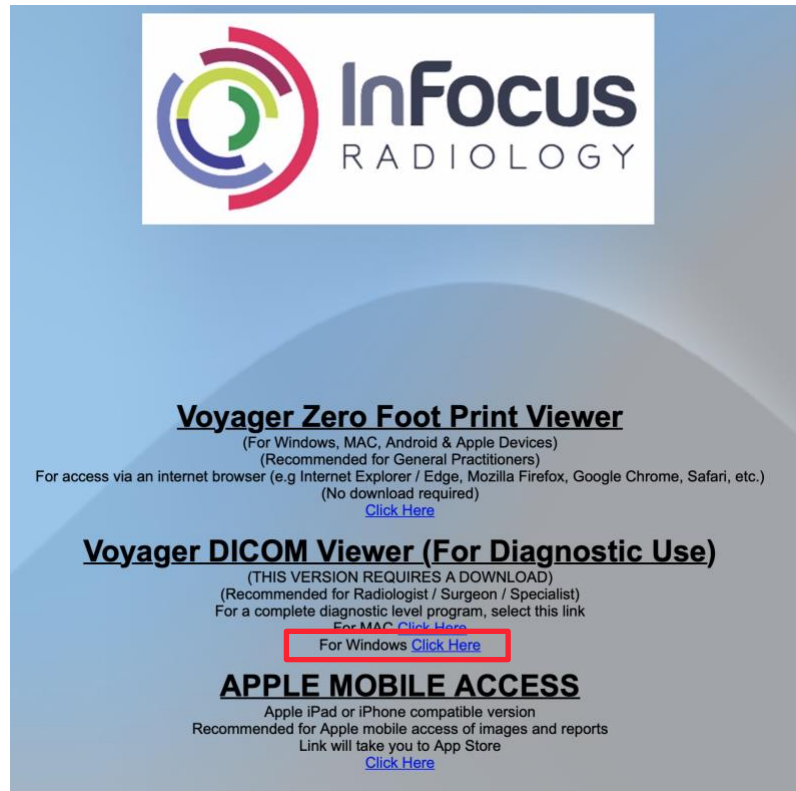


InFocus
RADIOLOGY

Voyager Zero Foot Print Viewer
(For Windows, MAC, Android & Apple Devices)
(Recommended for General Practitioners)
For access via an internet browser (e.g Internet Explorer / Edge, Mozilla Firefox, Google Chrome, Safari, etc.)
(No download required)
[Click Here](#)

Voyager DICOM Viewer (For Diagnostic Use)
(THIS VERSION REQUIRES A DOWNLOAD)
(Recommended for Radiologist / Surgeon / Specialist)
For a complete diagnostic level program, select this link
For MAC [Click Here](#)
For Windows [Click Here](#)

APPLE MOBILE ACCESS
Apple iPad or iPhone compatible version
Recommended for Apple mobile access of images and reports
Link will take you to App Store
[Click Here](#)

2. On the new web page, click on 'Download Now' to download the VoyagerWebViewerSetup.exe.

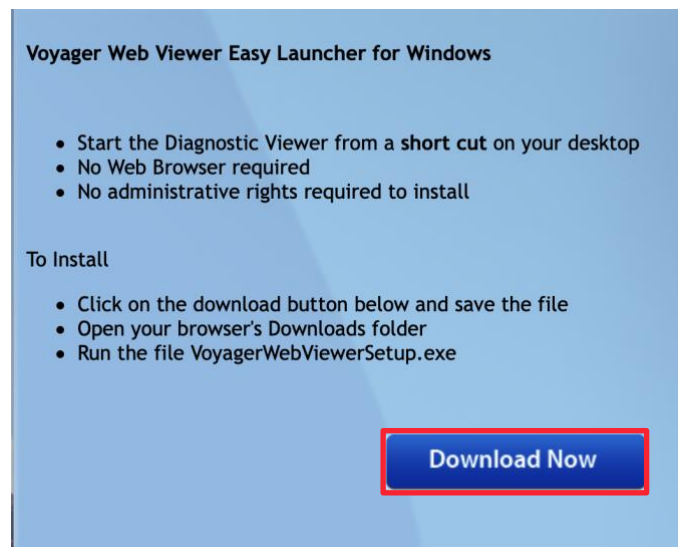

Voyager Web Viewer Easy Launcher for Windows


- Start the Diagnostic Viewer from a short cut on your desktop
- No Web Browser required
- No administrative rights required to install

To Install


- Click on the download button below and save the file
- Open your browser's Downloads folder
- Run the file VoyagerWebViewerSetup.exe

[Download Now](#)


3. Once the installer is downloaded, run it. (If you get a prompt 'the publisher of **VoyagerWebViewerSetup.exe** couldn't be verified', then click 'Run')
4. Voyager Web Viewer Setup Wizard will open up and click 'Next'.

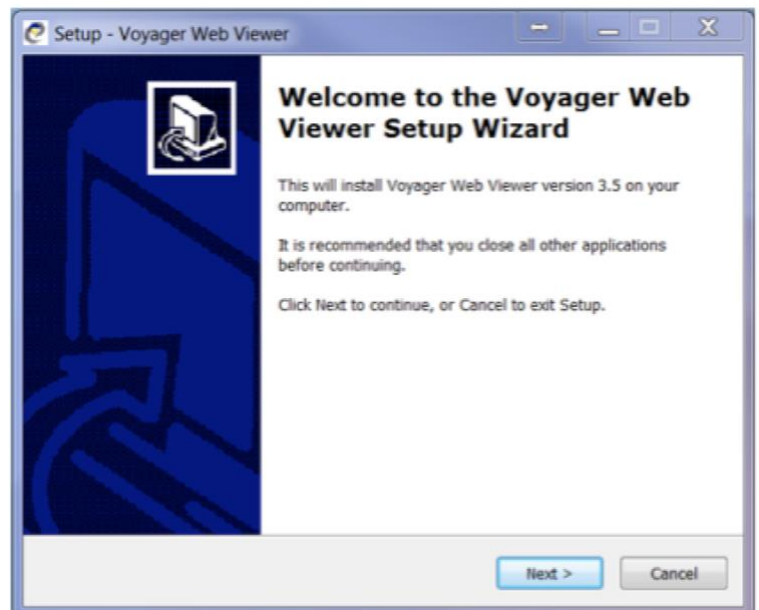

5. Under 'Select Destination Location', a default path will be displayed. Click 'Next' (If you would like to install at a different location, then please 'Browse' and select).

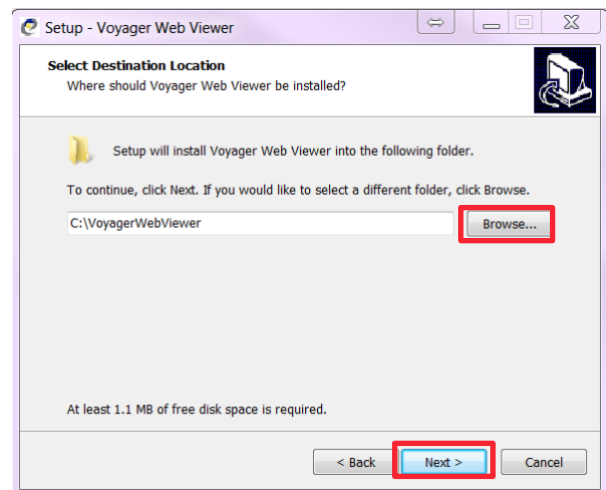

6. Under 'Select Destination Location', a default Start Menu Folder will be displayed. Click '**Next**' (If you would like to select a different folder, then please '**Browse**' and select).

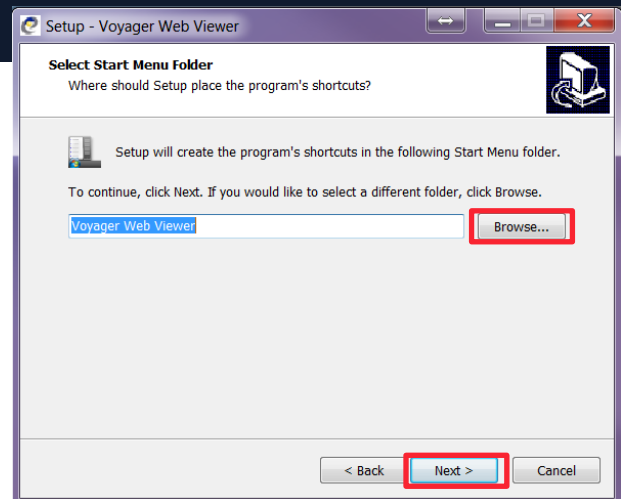

7. Under '**Select Additional Tasks**', check the box '**Create a desktop icon**' and click '**Next**'.
8. On the next view click on '**Install**'.
9. Once the installation is complete, click '**Finish**'.
10. Go to Desktop and double click on **Voyager Web Viewer icon**.
11. Voyager Web Launcher application will open up. Put the Server Name as – <https://images.infocusradiology.com.au> and hit '**Launch Web3**'.

Files will be downloaded from Iris Imaging's Server automatically and will launch Voyager PACS Web Viewer.

12. Enter username and password to log onto Voyager PACS Web Viewer.

- END -