


## Instructions to use Voyager Web on Mac

Please follow the steps below to login to Voyager using web:

1. Open Safari and in the Address Bar type in <https://images.infocusradiology.com.au> and hit Enter.
2. Click on the link '**For MAC Click Here**', under Voyager DICOM Viewer (For Diagnostic Use).

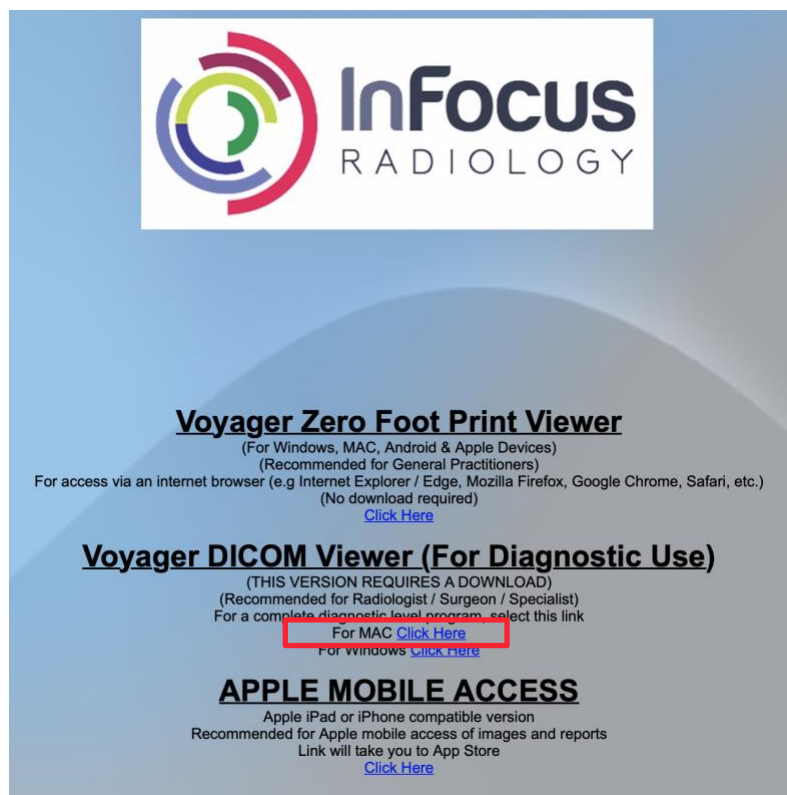

**InFocus**  
RADIOLOGY

**Voyager Zero Foot Print Viewer**  
(For Windows, MAC, Android & Apple Devices)  
(Recommended for General Practitioners)  
For access via an internet browser (e.g Internet Explorer / Edge, Mozilla Firefox, Google Chrome, Safari, etc.)  
(No download required)  
[Click Here](#)

**Voyager DICOM Viewer (For Diagnostic Use)**  
(THIS VERSION REQUIRES A DOWNLOAD)  
(Recommended for Radiologist / Surgeon / Specialist)  
For a complete diagnostic level program, select this link  
[For MAC Click Here](#)  
[For windows Click here](#)

**APPLE MOBILE ACCESS**  
Apple iPad or iPhone compatible version  
Recommended for Apple mobile access of images and reports  
Link will take you to App Store  
[Click Here](#)

### How to install and launch Voyager web3 on a Mac


#### System Requirements:

- Intel-based Mac
- 100 GB free space on Hard disk


#### Instructions to Install Web3 client

- Click here to download the [Voyager Imaging web loader](#)
- Run VoyagerImaging.dmg this will add 'VoyagerImaging' under Devices in Finder
- Run 'Voyager Imaging' from the VoyagerImaging device and confirm 'move to Application folder'
- Open the 'Applications' folder and drag 'Voyager Imaging' to the Dock bar
- After starting Voyager Imaging, Select or type in the web server name (i.e. 'SERVER NAME OR IP') and click the 'Launch web3' button.
- Voyager web client will load and launch


3. A new page will load up and click on '**Voyager Imaging Web Loader**' and the installer will start getting downloaded from the Server.

- Once the installer is downloaded, double click '**VoyagerImaging.dmg.zip**' in '**Downloads**'. This will add Voyager Imaging under Devices.
- Run Voyager Imaging and confirm to '**more to application folder**'.
- Open the '**Applications**' folder and drag '**Voyager Imaging**' to the Dock Bar.
- Go to '**Dock Bar**' and open '**Voyager Imagining**'. 

- Voyager Web Launcher application will open up. Put the Server Name as <https://images.infocusradiology.com.au> and click '**Launch Web3**'.

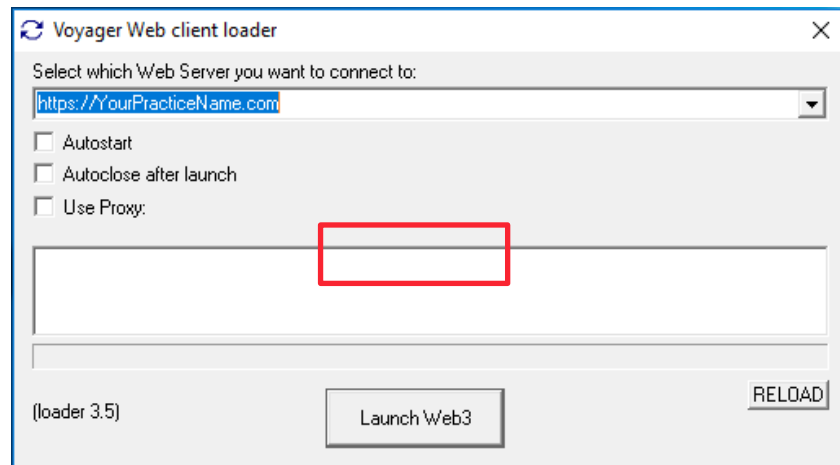

- Once Voyager Web loads up, put in your Username and Password and click '**Login**'.
- Voyager Web will log into the Server. Click on the relevant checkboxes on left or go to '**Quick Search**' on the top to find your patient's images.

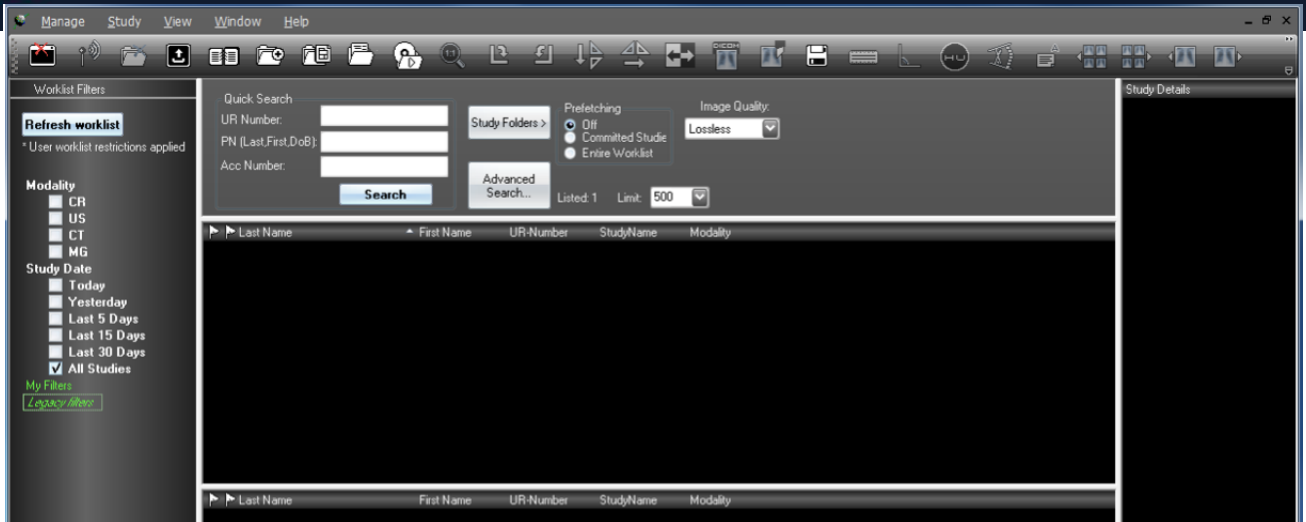

- END -

